



GENTLE CARE FOR YOUR YOUNGEST PATIENTS



MINICOLLECT® PIXIE Heel Incision Safety Lancet

MiniCollect® PIXIE is a safety lancet for the heel. With a shallow penetration depth, PIXIE nevertheless gently ensures maximum blood flow. This product is ideal for premature and new-born babies. Three different PIXIE safety lancets are available for heel incision, each adapted to a different stage of infant development. The depth of penetration and the length of incision differ depending on the selected device.

Thanks to optimal ergonomics and the fully enclosed blade, MiniCollect® PIXIE can enhance safety during capillary blood collection from the heel.

- / Ergonomic design and non-slip texture
- / Non-detachable safety tab
- / One-handed activation
- / Trigger positioned on side prevents tissue compression

PRECISE INCISION
THANKS TO TAPERED
LANCET SLOT AND
DUAL COLOUR.

When MiniCollect® PIXIE is positioned on the heel, the dual colour design, together with the arrows on both sides, allow a precise identification of the puncture site.



PRODUCT OVERVIEW

FOR FURTHER INFORMATION AND/OR SAMPLE ORDERING
PLEASE VISIT OUR WEBSITE OR CONTACT US.

MiniCollect® PIXIE Heel Incision Safety Lancet

sterile, with blade

Item No.	Colour coding	Incision length	Penetration depth	Inner / Outer [Qty.]
450525	● light-green	1.40 mm	0.65 mm	50 / 200
450526	● orange	1.75 mm	0.85 mm	50 / 200
450527	● yellow	2.50 mm	1 mm	50 / 200

RECOMMENDED HANDLING

Please observe the instructions for use at www.gbo.com.



1

Pre-warm and disinfect the puncture site according to your internal guidelines.



2

Press down on the safety tab to release the locking mechanism.



3

Position lancet, push trigger on the side and then dispose of PIXIE properly.



4

Gently apply pressure to the surrounding tissue and discard the first drop of blood.

NOTE: Please follow the recommendations of CLSI (GP42, 7th Edition), on selection of the appropriate lancet for heel incision, taking into account the baby's weight and age.